## ZAMST RK-1 Plus (KNEE SUPPORT) INSTRUCTIONS FOR USE

EN

For safety reasons, be sure to read this user manual before use. Also, store it in a location where you can refer to it at any time.

### Read this first:

- This product has a different design for the left and right legs, and is designed to be used when you are experiencing pain in the outer sides of one or both knees. If you are experiencing pain in the inner side of your knee, apply the product designed for the right knee to your left knee, and vice versa.
- Support may not be provided by this product under certain usage conditions.
- This product is not intended to be used for medical treatment.

### When using the product:

- Due to the characteristics of the materials used in this product, the color may change depending on storage conditions.
- When storing the product, avoid applying excessive weight to the hook-and-loop fasteners or raised fabric. Otherwise, the hook-and-loop fasteners may not work effectively.

### Precautions for use - Be sure to read this:

Safety precautions are divided into the following sections, according to the level of danger involved.

**⚠ WARNING:** Indicates a potentially hazardous situation, which may result in serious personal injuries if not avoided.

**⚠ CAUTION:** Indicates a potentially hazardous situation, which may result in minor personal injuries and/or property damage if not avoided.

### ⚠ WARNING

- To prevent abnormalities or worsening of symptoms, people who suffer from the following conditions should refrain from using the product:
  - Those predisposed to allergies or with sensitive skin.
  - Those who have a fracture, cut, numbness, swelling, eczema, rash or other abnormalities in the area where the product will be worn.
- Be sure to follow the warnings below and use the product correctly. If the product is not used correctly, it may cause circulatory problems or injury.
  - The product should only be used on knees.
  - Be sure to use the correct size product.
  - Do not use the product while sleeping.
  - Follow the attachment steps in order to use this product correctly.
  - Do not wrap the main body of the product or the straps too tightly.
  - If the product loosens or comes off during use, be sure to refasten it correctly by following the noted steps from the beginning. If wearing the product for a long period of time, take extra care to maintain appropriate compression force, and readjust the tightness as necessary.
  - If the product shows any signs of wear or damage, or a hook-and-loop fastener no longer works well, do not use the product.
  - Do not attempt to modify or repair the product.
- If any abnormalities, such as an injury, fracture, pain, numbness, swelling, eczema, etc. occur when using this product, immediately stop using it, and consult a physician. Continued use may worsen symptoms.

### ⚠ CAUTION

- Hard materials are used for the hook-and-loop fasteners. In certain rare instances, fingers or skin may be damaged, so use the product carefully.
- Sweat and friction may cause the color of this product to fade or be transferred to other materials.
- Make sure the hook-and-loop fasteners not attach to clothing. If they do, clothing may become frayed or be damaged.

## Care and cleaning


### ⚠ CAUTION

- Do not wash this product with light colored items. By doing so, color may fade or be transferred to other materials.
- If washing the product together with clothing, etc., be sure to use a laundry net in order to prevent the clothing from becoming frayed or be damaged.
- This product can be washed in a washing machine. After washing, hang it to dry in the shade.
- When washing this product, attach the hook-and-loop fasteners to the main body. Attempting to wash this product without closing the hook-and-loop fasteners may weaken its fastening capability.
- Do not use an iron or chlorine bleach, however, oxygen bleach can be used.
- Do not tumble dry. Doing so may damage the product.
- This product cannot be dry cleaned. Wet cleaning is allowed, however, avoid tumble drying.


## How to apply

- Before attaching this product, carefully read the "Precautions for use - Be sure to read this" section. Then follow the steps provided to ensure the product is correctly attached.
- The steps below show how to use the product when you are experiencing pain in the outer side of your right knee. Follow the same steps for the left leg or when you are experiencing pain in the inner side of your knee.


1. With the Zamst logo positioned under your knee, slide the product so that the curved edge of the lower pile loops is positioned 1 cm away from your patella.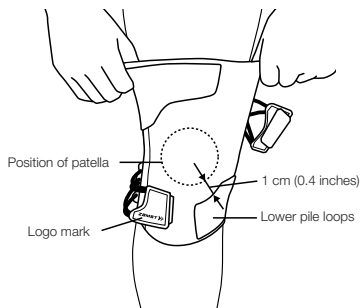

2. Straighten your knee and pull hook-and-loop fasteners A and B.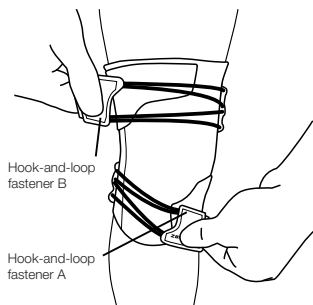


3. Tighten and fasten hook-and-loop fastener A so that appropriate compression is obtained.

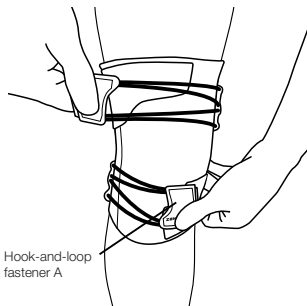
4. Tighten and fasten hook-and-loop fastener B so that appropriate compression is obtained.

5. While readjusting the compression of the strap, adjust the part that supports the back of your knee so that it is aligned with the center of the back of your knee.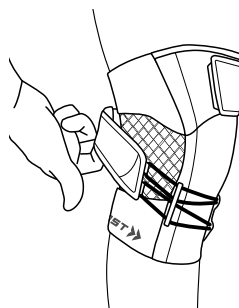

### The product correctly fastened

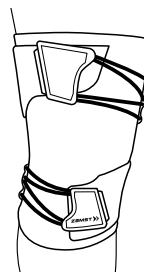Knee support for the right leg applied to the right leg  
(If you are experiencing pain in the outside of your right knee)

6. Make sure that the product is not loose or out of place. If you feel any discomfort, repeat the steps from the beginning to reattach the product.

### Disclaimer:

\* The product specifications and appearance are subject to change without notice for the purpose of improvement.

\* When disposing of this product, be sure to follow all disposal ordinances and regulations as specified by the local government.

\* We take all possible measures to ensure product quality, however, if you find any defects, etc. in our products, please contact Nippon Sigmex Co., Ltd.

"ZAMST" and "ZAMST logo" are trademarks or registered trademarks of Nippon Sigmex Co., Ltd in Japan, the United States and other countries.

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